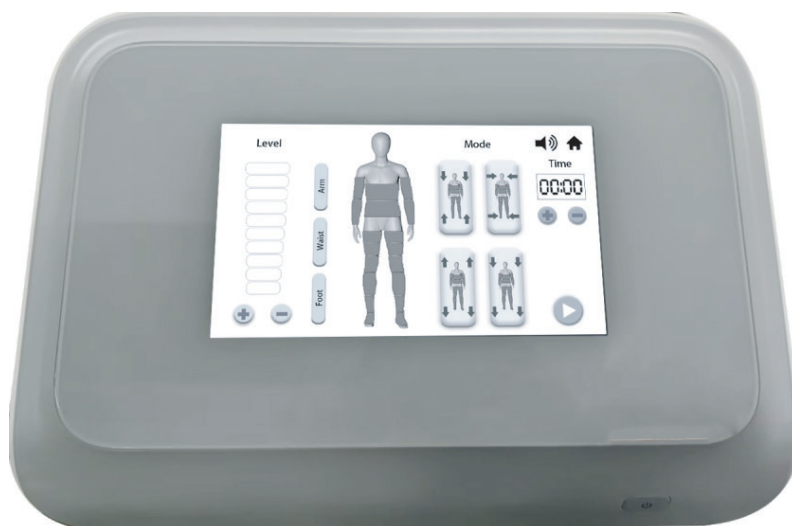
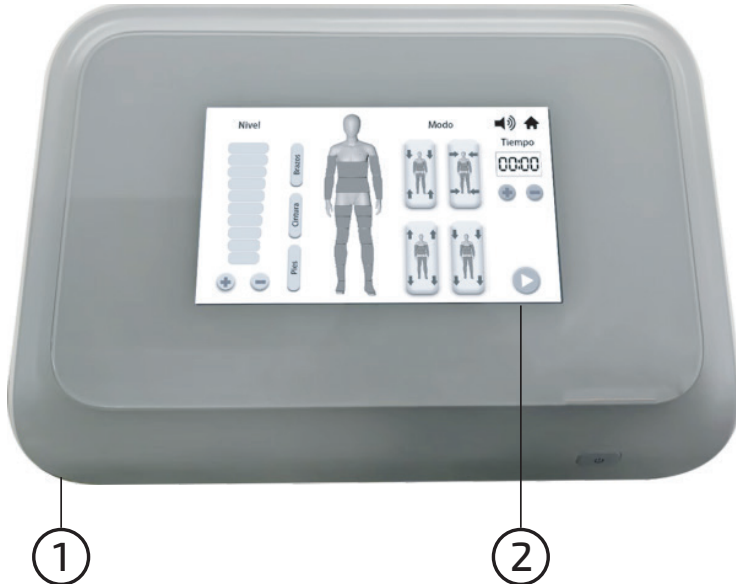
Instrucciones de operación  
Operation instruction  
Mode d'emploi

Panel de instrucción / Instruction panel  
Panneau d'instructions / Pannello di istruzioni

## ES

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EN - Connect the air pressure pipe to the host.  
After connecting the machine, lock the pipe and tighten the screws  
to prevent the pipe from coming loose.

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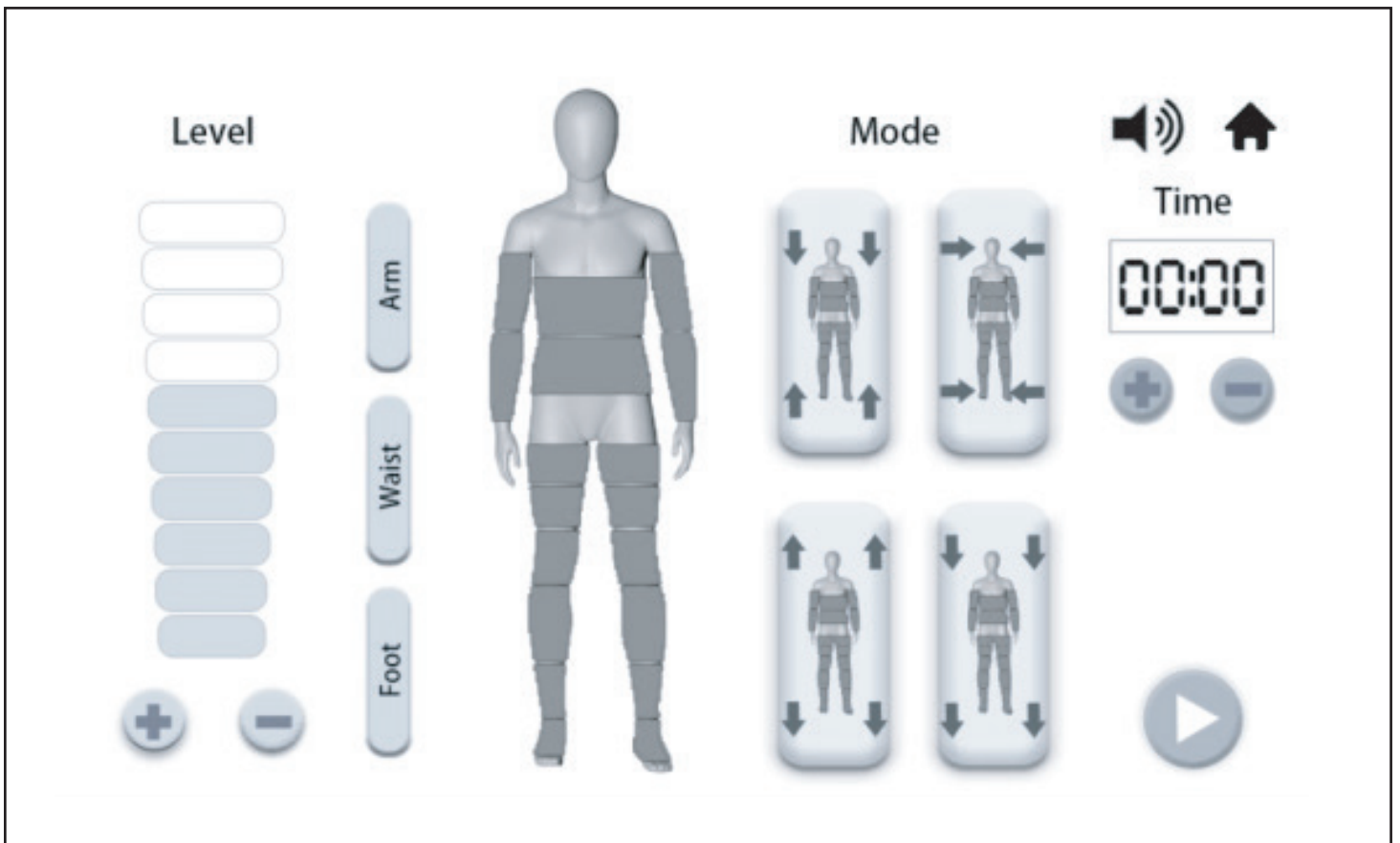
EN - Connect the power cord, turn on the power switch, and press the button Ⓞ  
Please press the button ▶ to start operation and the button Ⓜ to stop it.  
The time can be adjusted (preset to 30 min).

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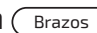
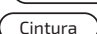
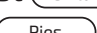
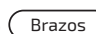
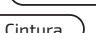
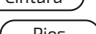
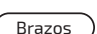
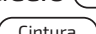
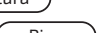
### EN

1. After entering the inflation interface, you can click the desired button.
2. Press this button  and choose the mode, then start operation.
3. "+" / "-": Adjust the suction level and relaxation time.
4. Press this button for the arm   
Press this button for the waist   
Press this button for the leg 
5. Select the body button; it will turn gray and stop working.

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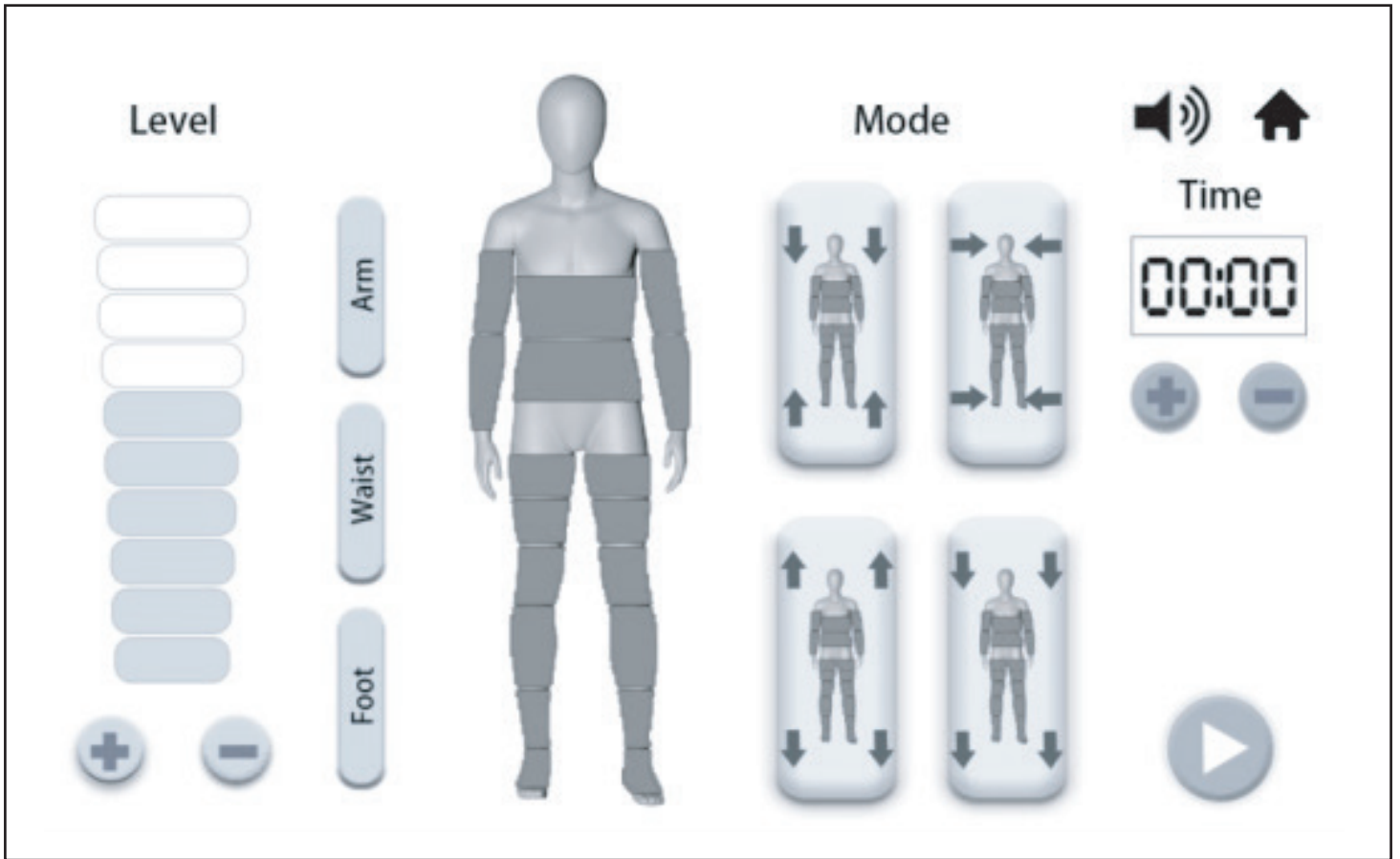
Press this button  and choose the mode, then start operation.

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### EN

1. Please do not disassemble the equipment or attempt any action not indicated in these instructions. All repairs must be carried out by our authorized professional staff.
2. Please do not assemble or operate this equipment near water or with wet hands. Do not pour any liquids on the equipment.
3. Please unplug the equipment under the following conditions and contact professional maintenance service:
  - 1- The equipment has come into contact with liquid.
  - 2- The equipment emits abnormal odors, smoke, or loud noises.
  - 3- The power cable is damaged.
  4. The equipment has been dropped and damaged.
4. Please do not place anything on the cables or place cables where they can be stepped on.
5. For your safety, unplug the power cable after use.
6. Do not insert anything into the equipment connector as it may cause fire or electric shock. If something enters the equipment, do not attempt to fix it yourself; contact the authorized distributor or our company immediately.
7. Do not place the equipment on unstable surfaces to avoid it tipping over and getting damaged.
8. Our company reserves the right to modify the information in this manual without prior notice. Please keep these instructions up to date.

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