Tratamiento de pressoterapia / Pressotherapy treatment / Traitement de pressothérapie

ES

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EN

With Nimbus, you can enjoy a high-quality professional pressotherapy treatment. This pressure massage machine features different modes to adapt the treatment to your needs, along with accessories designed to tone arms, legs, and abdomen. It also includes infrared heating and low-frequency EMS functions.

STRUCTURE

1. Fully equipped for complete body treatments: main unit, various pressotherapy accessories and electrodes (20 chambers), together with a pressurized air circulation system. Controlled via a touch-screen control panel. Maximum pressure: 28 kPa.
2. Power: 30W 5A / AC 110-240 V / 50-60 Hz.
3. Three available programs: electrostimulation, thermotherapy, and pressotherapy. *(Thermotherapy and pressotherapy cannot be used simultaneously.)*
4. SAUNA function through an infrared system.
5. EMS function: treatment using low-frequency electrical currents applied through electrodes.
6. Allows adjustment of all machine parameters up to 8 levels (temperature, pressure, exposure time, etc.).
7. Easy assembly and installation.
8. It is not recommended to use the full-body pressotherapy function all at once.

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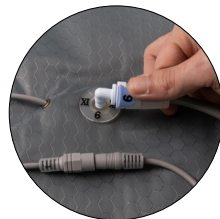
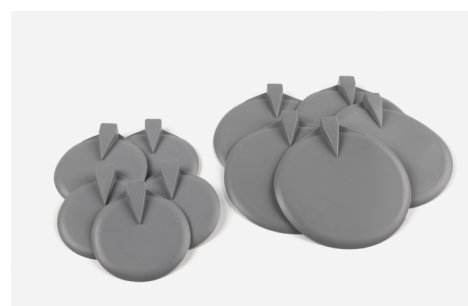
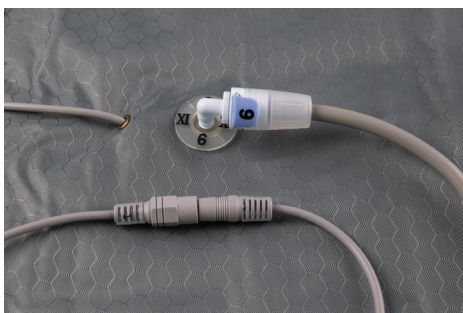
It is a pressure designed for experienced users or advanced clinical/sports applications, such as intense muscle recovery or deep drainage. It offers an adjustable range from low pressures (40–60 mmHg) up to 210 mmHg, allowing safe progression.

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Maximum pressure:	210 mmHg (28 kPa)
Number of chambers	20
Included areas:	2 legs + 2 arms + 2 feet + 1 abdomen
Equipment type:	Full-body pressotherapy system
Recommended profile:	Professional / Advanced Full-body treatment

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Main application

Air Pressure Function:

Pneumatic pressotherapy with sequential chamber compression. Lymphatic drainage, edema reduction, and relief of heavy legs.

EMS Function:

Electrical Muscle Stimulation (EMS). Toning, active recovery, and muscle strengthening without impact.

Physio Sauna Function:

Infrared sauna or controlled dry heat therapy (physiotherapy). Deep thermotherapy, muscle relaxation, and toxin elimination through perspiration.

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Competitive advantage

Therapeutic synergy:

Heat from the Physio Sauna system promotes blood vessel dilation, pressotherapy supports fluid circulation, and EMS activates the muscles. When used together, they enhance treatment results.

Space saving:

A single device replaces three independent systems.

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EN

INFORMATION

Read all instructions before beginning assembly. Keep the instructions for future use. Indoor use.

WARNING

Use only as intended.

ASSEMBLY

Follow the attached assembly instructions.

MAINTENANCE

Periodic maintenance tasks should be performed by checking the condition of the product. Ensure the product is in good condition and functioning properly. Check for breaks regularly. Remove the product from service when any condition that could be considered unsafe occurs.

CARE AND CLEANING

Clean the unit with a damp cloth or sponge using warm water and mild soap. Dry thoroughly. Do not use abrasive cleaners, bleach, chemical solvents, or strong detergents.

WARRANTY

Please refer to the General Terms and Conditions of Sale

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