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
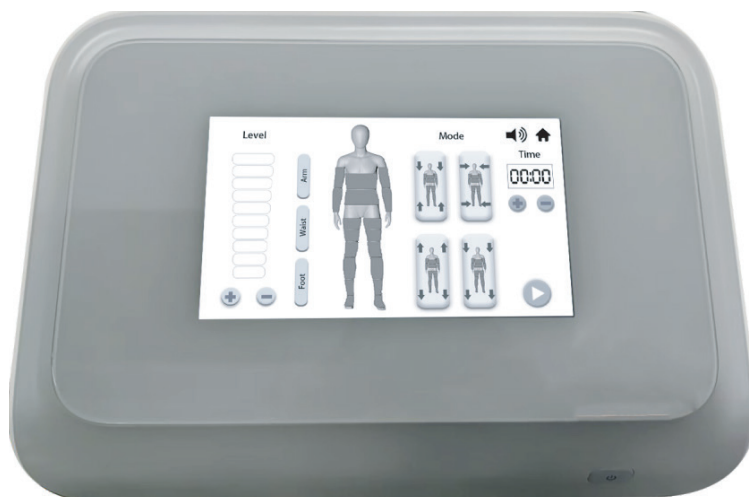
Operation instruction

Mode d'emploi

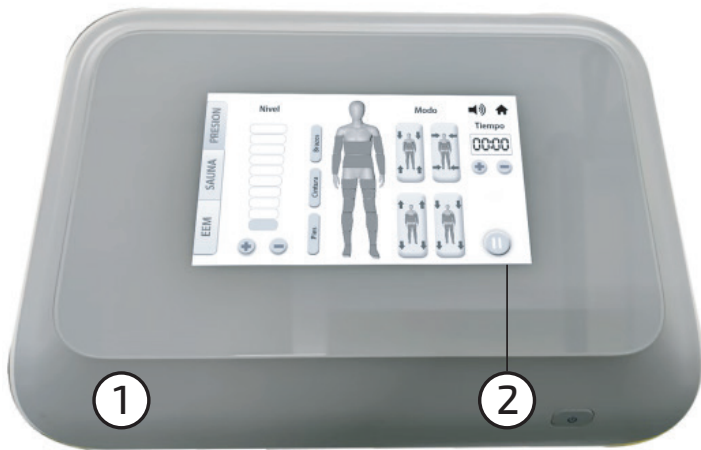Panel de instrucción / Instruction panel
Panneau d'instructions / Pannello di istruzioni

ES

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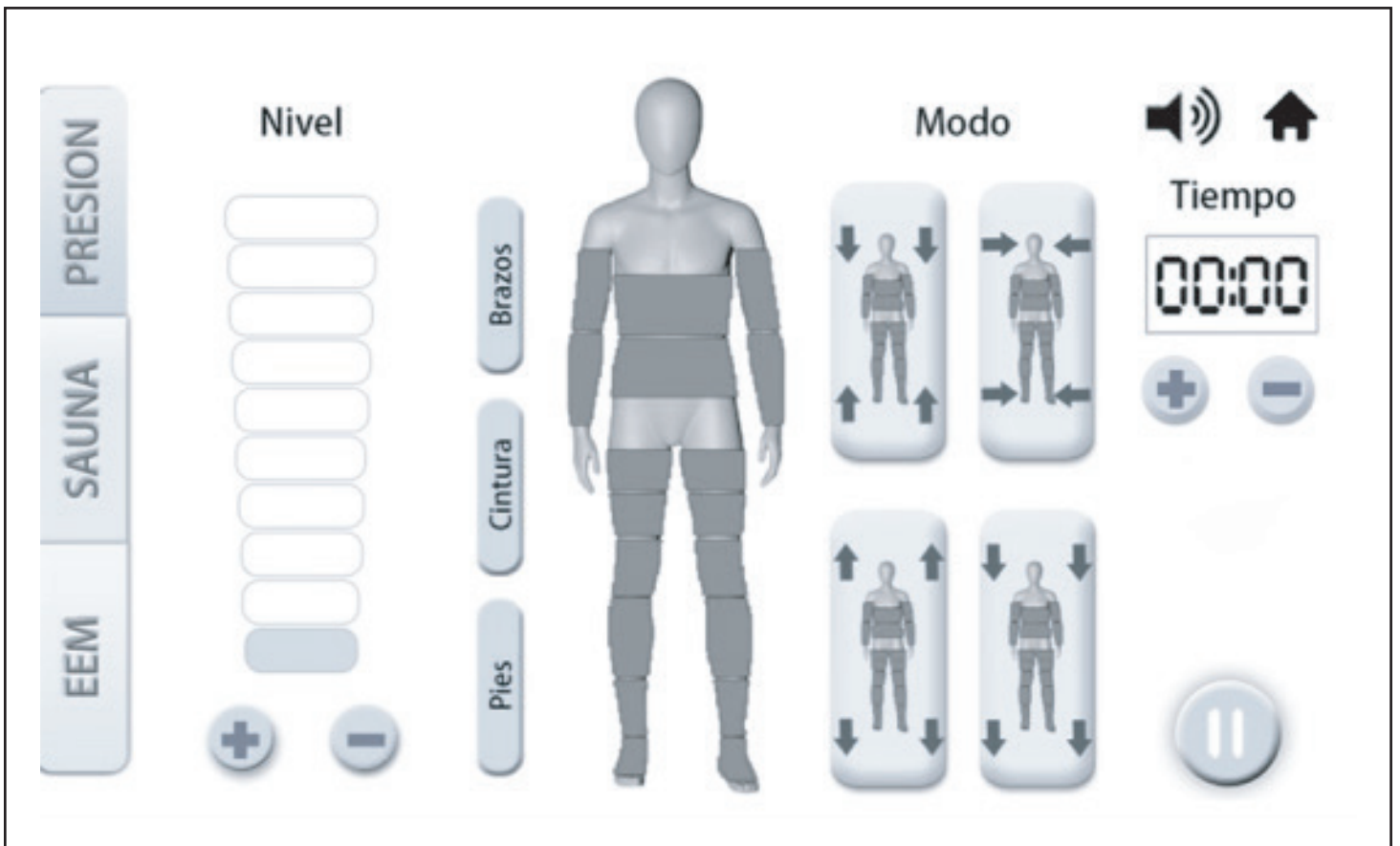
EN - Connect the power cord, turn on the power switch, and press the button Ⓢ. Please press the button ▶ to start operation and the button Ⓜ to stop it. The time can be adjusted (preset to 30 min).

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EN

1. After entering the inflation interface, you can click the desired button.
2. Press this button  and choose the mode, then start operation.
3. "+" / "-": Adjust the suction level and relaxation time.
4. Press this button for the arm 
- Press this button for the waist 
- Press this button for the leg 
5. Select the body button; it will turn gray and stop working.

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EN - Basic Guide

1. Air pressure massage (15-30 minutes)

1. Please wrap the client with preservative film on the body area to be treated. Then the client should wear the slimming cloth accessory and lie down on the facial bed.
2. Start the machine by pressing the power button.
3. Pause the output of electrostimulation and infrared heating.
4. Press the output start/stop button.
5. Adjust the air pressure mode and time according to the client's capacity.

2. Infrared heating for 20-30 minutes

1. Wrap the client with a wet towel on the body area to be treated. Then the client should wear the slimming cloth accessory and lie on the facial bed.
2. Start the machine by pressing the power button.
3. Pause the output of electrostimulation and air pressure.
4. Press the output start/stop button.
5. Set the heating intensity to 8; when the temperature gets high, adjust the intensity to 5/6 to maintain temperature.

3. Electrostimulation for 10 minutes

1. Start the machine by pressing the power button.
2. Pause the output of air pressure and infrared heating.
3. Press the output start/stop button.
4. Select the desired output and adjust the intensity.
5. You can choose manual or automatic mode. When the red light flashes, it is manual mode; otherwise, it is automatic. There are 5 modes in manual mode.
6. Electrode pad positions (see images):

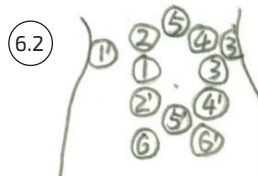
6.1 For shoulder and neck

Function: Improve blood circulation in shoulder and neck; reduce fat in these areas; relieve tension.

6.2 For shoulder and arm

Function: Improve blood circulation in shoulder and arm; reduce fat in arms.


6.3 For abdomen

Function: Improve blood circulation in abdomen; reduce fat in the area.

6.4 For thigh

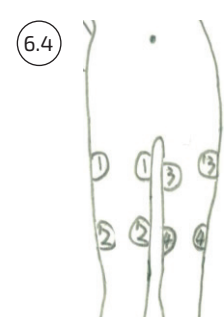Function: Contract thigh muscles; reduce fat in the thigh.

4. You can combine air pressure, infrared heating, and electrostimulation treatments.

5. It is recommended to shower after treatment.

6. The recommended total treatment time is 90 minutes.

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EN

1. Please do not disassemble the equipment or attempt any action not indicated in these instructions. All repairs must be carried out by our authorized professional staff.
 2. Please do not assemble or operate this equipment near water or with wet hands. Do not pour any liquids on the equipment.
 3. Please unplug the equipment under the following conditions and contact professional maintenance service:
 - 1- The equipment has come into contact with liquid.
 - 2- The equipment emits abnormal odors, smoke, or loud noises.
 - 3- The power cable is damaged.
 4. The equipment has been dropped and damaged.
 4. Please do not place anything on the cables or place cables where they can be stepped on.
 5. For your safety, unplug the power cable after use.
 6. Do not insert anything into the equipment connector as it may cause fire or electric shock.
- If something enters the equipment, do not attempt to fix it yourself; contact the authorized distributor or our company immediately.
7. Do not place the equipment on unstable surfaces to avoid it tipping over and getting damaged.
 8. Our company reserves the right to modify the information in this manual without prior notice.
- Please keep these instructions up to date.

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