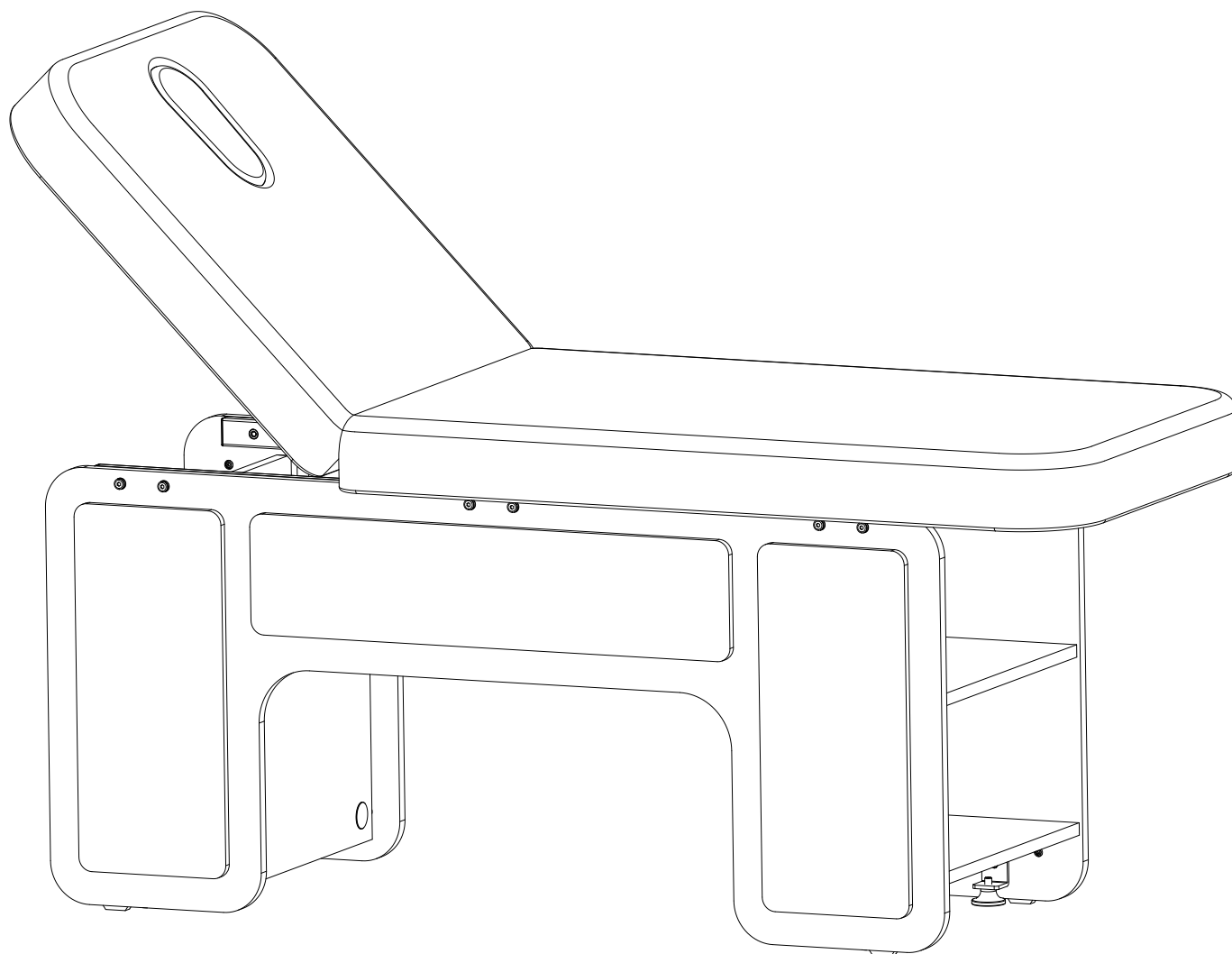
Instrucciones de operación
Operation instruction
Mode d'emploi

INDUS

Welcome to **INDUS** beauty bed!
Before using this electric beauty bed, please read
the operation instruction carefully.

ES	EN	FR	
INDICE	INDICE	INDICE	
Composición estructural de la cama de belleza eléctrica	Structural composition of electric beauty bed	Composition structurelle du lit de beauté électrique	Pg 3
Ámbito de aplicación	Scope of application	Champ d'application	Pg 3
Peligro	Warning	Avertissement	Pg 4
Parámetro técnico	Technical parameter	Paramètre technique	Pg 5
Montaje de piezas	Assembly of parts	Assemblage de pièces	Pg 6~8
Mecanismo de ajuste	Adjusting mecanism	Mécanisme d'ajustement	Pg 9
Instrucciones de seguridad importantes	Important safety instructions	Consignes de securite importantes	Pg 10
Mantenimiento del Producto	Product Maintenance	Entretien du Produit	Pg 10
Mantenimiento diario	Daily maintenance	Entretien quotidien	Pg 10
Sugerencia de uso	Suggested use	Usage suggéré	Pg 11
Limpieza del Producto	Cleaning of the Products	Nettoyage du Produit	Pg 11

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EN

PARTS OF ELECTRIC BEAUTY BED

This beauty bed is composed of 1 gas spring, hardware, wooden parts, and leather surface.

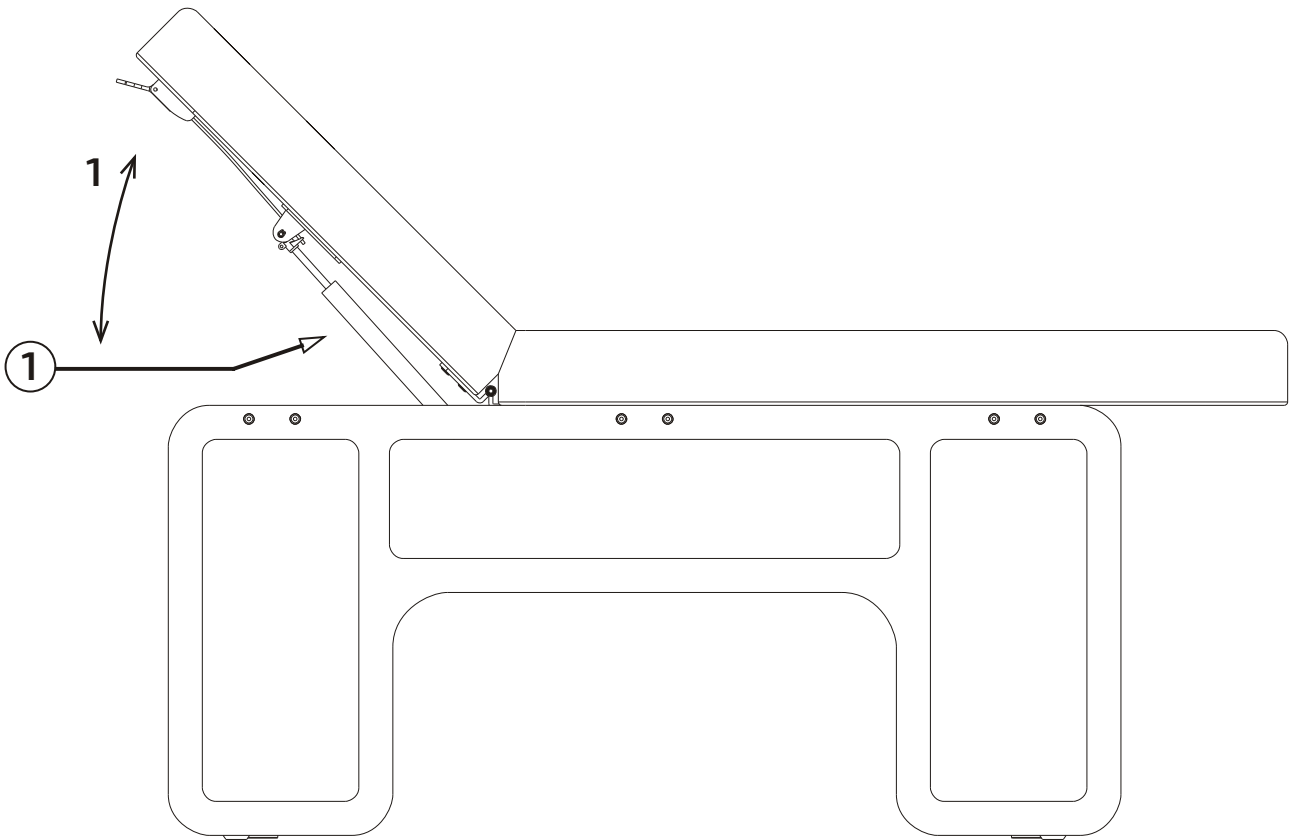
This beauty bed includes a backrest angle control part, which is an integral integrated structure.

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SCOPE OF APPLICATION

This electric beauty bed is suitable for nursing or providing beauty and massage services to customers.

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EN

⚠ WARNING

The power must be out off when performing maintenance or cleaning the machine

The plug must be fully inserted into the socket to prevent fire or electric shock.

Do not stand on this equipment.

Keep away from the reach of children. Do not allow children to play with the equipment

Please do not place the equipment in damp places such as bathrooms to avoid electric shock or malfunction; The remote control should also avoid contact with water.

Do not operate this equipment in areas where painting/spraying or oxygen supply is in progress.

Only experienced professionals are allowed to use this equipment

If the plug is damaged, please contact the manufacturer, agent, or skilled personnel for replacement to avoid danger.

During the operation of the beauty bed, ensure that there are no touchable items within the range of movement.

This product is only suitable for use by beauty professionals with professional qualifications.

When pressing the buttons on the hand or foot controller to adjust the backrest angle and the bed height, it is prohibited to place hands, feet, head, and other parts in position of the diagram ① (see the following figure for details).

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EN

IMPORTANT SAFETY INSTRUCTIONS

The product's cable cannot be stretched, tightened, or torn, because the product may stop working.

Do not shake the product, do not hit or the product may tip over and fall on users; causing damage to the product and/or injury to the Users.

The voltage of the product is 110v/220v, please choose the appropriate voltage to avoid burning the electrical parts.

When using the product, the maximum current can reach 10A or higher. Use high-power connections.

If possible, use an individual plug, to avoid power failures or fire due to overload.

If the cable is damaged, change it or contact the supplier for after-sales service.

Do not place fuels near the product.

Do not place objects of a different color than the surface, they may stain the skin of the product.

Do not spray water to clean the product, otherwise, it may cause product leakage, short circuits and may lead to electric shock or damage to the product. And that it stops working.

Do not expose the product to the sun, otherwise the color of the product may vary.

Unplug the product when it is not going to be used for a long period of time.

PRODUCT MAINTENANCE

Turn off the power off when product is not in use.

Do not stack sundries above the product, do not use sharp utensils access products.

The product must be placed in the dry environment instead of humid.

The product can not withstand the impact, so do not place heavy objects over 100Kg above the product.

The limit load of the product is 200Kg, using the product in safety load of 150Kg.

When the product does not work, please contact the supplier!

DAILY MAINTENANCE

After each session, use neutral reagents to remove dust and stains from the leather and wooden parts to prevent premature aging of the leather and wooden parts.

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EN

SUGGESTED USE

Before connecting, operating or adjusting this electric beauty bed, carefully read the following instructions and keep this manual safe.

This electric beauty bed uses a three wire power cable. When using it, it must be plugged into a three hole power socket with a ground wire to ensure that the electric beauty bed is properly grounded to prevent accidental electric shock for users, and to ensure the normal operation of the electric beauty bed.

When performing the direct control of the beauty bed's motor lifting operation, the control button must be continuously pressed until the desired position is released.

When using the electric push rod of this electric beauty bed, it should be noted that the frequency of use is Max 6min/h (Continuous operation within 1 hour cannot exceed 6 minutes)

Repairs must be carried out by trained professionals.

After each nursing session, use neutral reagents to remove dust and stains from the skin and decorative parts.

CLEANING OF THE PRODUCTS

Cut off power when cleaning product.

When cleaning product, remember to remove other item in product to prevent slipping down during the cleaning process, resulting in injuries and damaged goods;

Do not spray water to clean the product, otherwise, it may produce a short circuit, or may produce some electric shock.

Use a special cleaning agent when cleaning, oily liquid can not be used, or will cause the leather contamination, corrosion and other phenomena.

Do not use cleaning agent with a liquid dye when cleaning, or will cause the damaged to the local color of the leather ;

Do not use cleaning agent with corrosive liquid, or will cause the damaged to the plastic and leather .

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EN

IMPORTANT INFORMATION

Read all instructions before beginning assembly. Keep instructions for future use.

SUGGESTED USE

The seat is designed for ages 6 and up. Movable, ideal for the flexibility of the classroom, common spaces. Allows for multiple configurations for collaborative work environments. Indoor use.

WARNING

Use only as intended. Do not attempt to stand or kneel on item. Do not stack just flat join.

ASSEMBLY

Follow instructions of attached assembly.

MAINTENANCE

Regular maintenance tasks have to be realized by checking the product's conditions. It is important to check that the product is in good conditions and working properly. Check for breakage periodically. Remove the product from service when any condition develops that might deem operation unsafe.

CARE AND CLEANING

Clean the unit with a damp cloth or sponge using warm water and mild soap. Wipe dry. Do not use abrasive cleaners, bleach, chemical solvents, and strong detergents.

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For possible future updates to the document, visit this QR link to see the latest version.

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